

Otovax Spray

Medical device

Medical device OTOVAX SPRAY is a safe product for regular hygiene treatments which contains ingredients of natural origin, that help to keep the proper ears hygiene.

It melts and removes the excess of earwax and builds protective layer with hydrophobic properties protecting ear canal against action of water. It is used to keep the outer acoustic duct clean and dry .

Destination:

- softening and removing of the excess of earwax and prevention of gathering of earwax in the outer acoustic duct
- creation of protective layer against action of water in people with so-called "swimmer's ear".

Especially recommended for people:

- who swim in swimming pools/ water tanks
- with "swimmer's ear"
- wearing deaf aid

Preparation for adults and children above 3 years old.

Content:

Ingredients: light paraffin oil, triglycerides of medium chain fatty acids, turpentine oil.

How to use:

Spray 1-2 doses in the outer acoustic duct. The product should be applied in the sitting or standing position. Use 1-2 times a week to release ear canal from the excess of earwax or 2-3 times a day for 2 days to release from wax concretion.

Packing:

- 20 ml
- 400 of single doses.

